

PATCHES WITH CARBON PAD AND CARBON WOUND DRESSING



Patch 2,5 cm × 8 cm, steril.

- prevents infection
- reduces the feeling of pain
- sucks out toxins
- **Use:** For minor injuries, burns, widely used for treatment of patients after dialysis, suitable for wound treatment after tick removal.

Activate the carbon during application: with physiological solution, drinking water

ACTIVE CARBON WOUND DRESSING

Packed in food foil, sterile.

(carbon layer, yarn, non-woven fabric and gauze)

- Significant deodorizing effect
- Quick cleaning of wounds
- Treatment of non-healing wounds (leg ulcers, bedsores, bacterially threatened wounds,...)
- Treatment after surgical interventions
- Accelerated build-up of bleeding
- Removal or prevention of the spread of poisons
- Absorbs microorganisms
- It effectively prevents the multiplication of bacteria
- Eliminates Pseudomonas aeruginosa, Staphylococcus aureus, Escherichia Coli within 24 hours

The products are basically made from practically pure carbon microfiber (99.999%) at a high temperature. The use of carbon has been known in medicine for a long time, but only these products are one of the few that can be applied directly to wounds, thereby accelerating the excellent effects of carbon to the maximum. The special PoP/Vi support layer guarantees good suction of wound secretions, but at the same time ensures the mechanism of a suitable moist wound microclimate.

The products are intended for both human and veterinary use.

Tecasorb – heamostatic absorption cover – sterile Composition: carbon riber and nonvoem faintin made of a blind of polypropriete in viscow, pilore using viran adia abiosity attached conting guase compression paid. Site: The band is manufactured in site fox9 cm and the product is sterilized by amount adult, and the sterilized as a disposable dressing for acute and chronic vonds too. structions for use: Before use, check the integrity of the package. Open the acitage and remove the compress with there are between using disposition to for more the appoint with the carbon fiber on the wound. (It's important not to more the wound in the wound.)

9 cm × 10 cm

na to the range and hend or hully, following the instructions of the high physical. With chores is usually are recommend changing after 6 to 12 when the wound gets clear, after 24 hours iso. The analysis of the structure of the single use easily in setting the being omised to the single setting the single setting the setting of the single use in case of Arther use there is a risk of containitation. where C2: to one 4, 110 00 Prate 1 thatone com recessorb.cz

Tel.: +420 603 421 517

www.tactical-evo.com

E-mail: info@tactical-evo.com

REFERENCES:

Army of the Czech Republic, special forces in Afghanistan and Mali, Military Health Insurance Company of the Czech Republic, Israeli Special Forces, Indian Army. Supplies to companies for first aid kits, private individuals, sport clubs - rugby, mountaineers, cyclists - sets, carbon for treating abrasions + magnesium in a spray for spasms.

Foto laického ošetření:

Mr. Václav

non-healing wound in the foot for 3 years, after a month of using carbon the foot is in much better condition.

Mr. Martin

finger burn while cooking pot stew. It did not heal well, we applied a patch with a carbon pad.

Mr. Olda

at the cabin - woodworking, very sharp ax. Most people have an oil rag and a sock at the cabin for first aid treatment. Mr. Olda was equipped with a 9x10 cm carbon cover. He soaked the carbon cover with water, put it on the wound, covered it with a bandage and drank a bottle of beer. The hospital is far away from the scene. A medical professional (mother-in-law) who was in the area, after removing the bandage, judged that no further intervention was needed. Mr. Olda has a bit of a scar on his leg because the wound was not sewn up, but he is not worried about it, he still lives with his wife and two daughters and goes to the cottage.

In Australia, our carbon pad patches were tested by sheep shearers. When a sheep shearer scratches him/herself while shearing, an unpleasant sore appears in the affected area. They call it sheep shearer's disease. Carbon cleans the affected area, sucks out inappropriate things donated by the sheep, the wound heals faster and the carbon pad also suppresses pain in the area.

www.tactical-evo.com